


# 3 Steps for setting up the **BiPAD**<sup>®</sup> Hand Switch

BiPAD<sup>®</sup> Disposable Bipolar Forceps Cord with Integrated Hand Activation

Simply use our cord to replace yours.


Use *your forceps*.  
Use *your generator*.

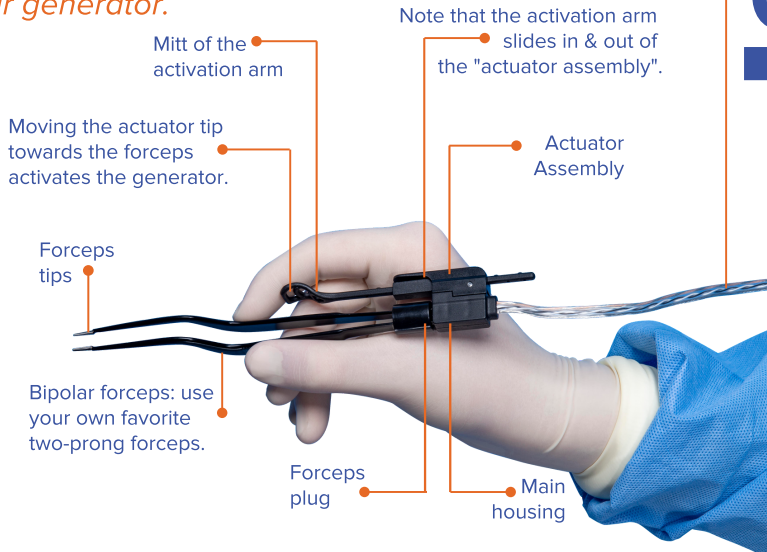

## STEP 01

## ATTACH

the BiPAD<sup>®</sup> disposable cord to your generator.

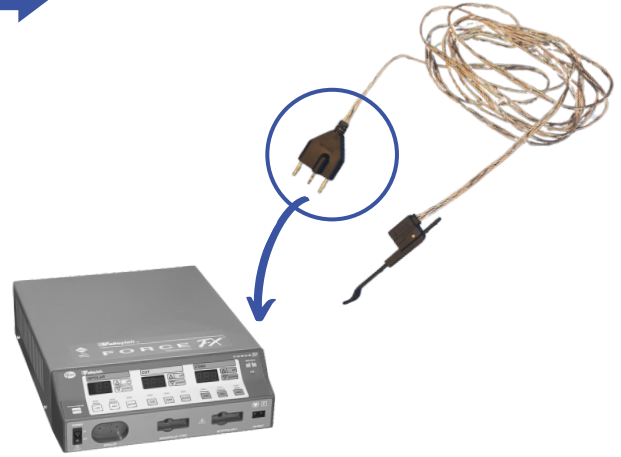


## ATTACH

*your forceps* to "main housing" - adjusting for right or left.

If you are RIGHT-handed, hold the BiPAD<sup>®</sup> plug with the actuator assembly to your RIGHT and insert the forceps (as in the diagram to right). Vice versa for the LEFT.

The forceps should fit snugly with minimal movement relative to plug when activating hand switch.

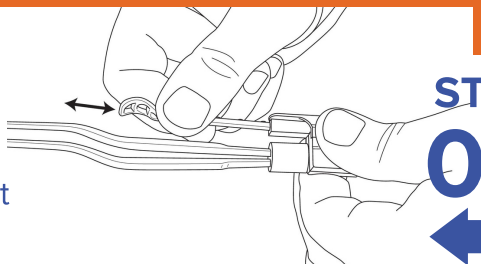

## STEP 02

## ADJUST


the length of the actuator arm.

HOLD the BiPAD<sup>®</sup> plug with one hand & HOLD the actuator assembly with the other hand.

Adjust the external arm length such that the "mitt" fits comfortably for activation by your finger (4a) - or - touches the space between the distal and middle phalanx (4b)

The external arm can be adjusted by pushing/pulling the external arm through the actuator housing. Clicks will be heard as the arm is moved. Wiggle the arm (front to back) to fully seat prior to use.

Open and close the forceps with the tips of your fingers with out activating. Move your finger inward to activate.

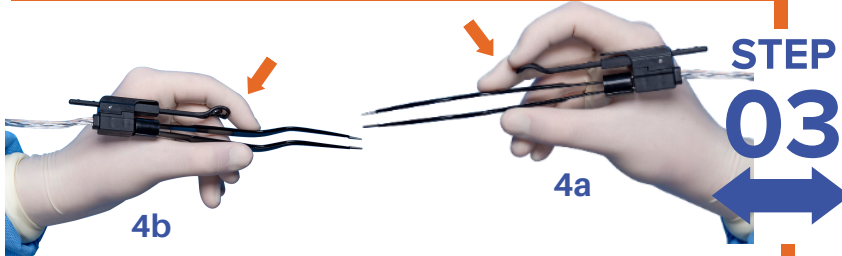

**NOTE: Foot Pedal Optional**

Simply remove actuary assembly & BiPAD<sup>®</sup> cord will activate the foot pedal.



small change  
**BIG IMPACT**  
Make the "switch" NOW.<sup>™</sup>

